# CARE™

- + Health & Grooming Kit
- + Kit Santé & Soins
- + Kit de Salud y aseo
- + Gesundheit & Körperpflege-Kit


**Instructions**  
**Instructions**  
**Instrucciones**  
**Anleitung**

boon<sup>®</sup>

**⚠ WARNING:**

- This product is not a toy. Do not allow child to play with it.

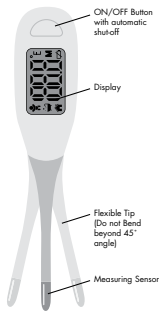**THERMOMETER:****⚠ WARNING:****BEFORE USING DIGITAL THERMOMETER, PLEASE READ CAREFULLY**

- This is NOT an ear thermometer. NEVER put probe into ear. Take temperature measurements Underarm, Orally or Rectally.
- NEVER allow children to play with this product. It is not a toy. This is a sensitive electronic device and may be damaged if not used properly under parental supervision.

- ALWAYS supervise your child when using thermometer. NEVER allow your child to use it alone or without close parental supervision.
- Avoid sudden impact and dust. Do not expose the thermometer to extreme temperatures, humidity, direct sunlight or shock.
- Do not use rechargeable batteries;
- Batteries must be removed and inserted under adult supervision;
- Non-rechargeable batteries are not to be recharged;
- Batteries are to be inserted with the correct polarity;
- Exhausted batteries are to be removed from the thermometer and disposed properly;
- Supply terminals are not to be short-circuited;
- Different types of batteries (e.g. alkaline, standard carbon-zinc or rechargeable) or new and used batteries are not to be mixed;
- Do not dispose of batteries in fire, may explode or leak.

**IMPORTANT INFORMATION:**

- Changes or modifications to this unit not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.
- Temperature readings are affected by many factors including exercise, drinking hot or cold beverages prior to taking a reading as well as technique.
- Do not allow children to walk or run during temperature taking. Do not allow child to chew on or bend the thermometer. The probe could present a hazard.
- Rectal temperatures vary by approximately 0.9°F/0.5°C.

- High, prolonged fever requires medical attention, especially for young children. Please contact your physician.
- Performance of thermometer may be degraded if operated or stored outside stated temperature and humidity ranges or the patient's temperature is below the ambient (room) temperature.

## SPECIAL PRODUCT FEATURES:


- Fast 8-second oral and rectal readings
- Flexible tip for added comfort
- Bath water temperature measurement
- Ambient temperature measurement
- Last Memory recall
- Automatic shut-off after approximately 5 minutes for longer battery life
- Replaceable battery
- Low battery indicator "⚡".
- Waterproof probe for easy cleaning

## MEMORY FUNCTION:

The thermometer memory function stores 1 measurement. The last measurement will appear after mode switched and no pressing for 3 seconds. For Body Mode, the last measurement will appear with green backlight (temperature lower than 99.5°F/37.5°C) or red backlight (temperature higher than 99.5°F/37.5°C) for 1 second.

## REPLACING THE BATTERY:

When "⚡" appears in the lower right of the display, the battery is exhausted. Using a coin or screwdriver, turn the battery cover counter clockwise to unlock and remove cover. Remove the old battery. Be aware of any local regulations concerning disposal of batteries. Place a 3V "CR2032" lithium battery inside with the positive side facing up. Place cover onto thermometer and rotate clockwise to lock in place.

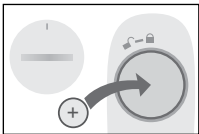## CAUTION:

Replace only with 3V "CR2032" lithium battery or equivalent. Use of other types of batteries may present a risk of fire or rupture.

## ⚠ WARNING:

- Keep battery out of child's reach. Swallowing battery could be fatal. Battery should not be charged or placed into extreme heat as it may explode.
- Used batteries should be disposed of properly.

## CHANGING FROM CELSIUS TO FAHRENHEIT:

- While thermometer is OFF, press and hold the ON/OFF button for about 5 seconds.
- Release button and you will see display (as shown).
- Press it again to switch between °F or °C. Power off automatically after switch completed.

## CLEANING AND CARE:

- Clean the thermometer tip and probe using isopropyl alcohol or soap and water before and after each use. DO NOT immerse in water over 3ft (91.4cm).
- Store the unit in its storage case while not in use.

- Make sure the thermometer has been at room temperature for 15 minutes before use.
- To avoid damage to probe, do not bend beyond 45°.
- Do not drop the thermometer as this may ruin the unit.

## MEASURING:

NOTE: Be sure probe tip is clean before using. See Care and Cleaning.

While thermometer is OFF, press the ON/OFF button to enter to the mode selection between Body, Water and Ambient mode. After mode selection completed, thermometer is ready for use.

Press the ON/OFF button. You will hear a short beep then all segments (as shown) will display for 1 second with red backlight for 0.5 seconds and then green backlight for 0.5 seconds. This is a function check indicating the unit is working properly. Then enter to the mode selection.

### 1. BODY MODE:

Display  and  $^{\circ}\text{C} / ^{\circ}\text{F}$  flashing with green backlight for 3 seconds, if no pressing within 3 seconds, a short beep sound is heard, then last memory will display and then display "--", "-.-", "-.-" circularly to start measuring. A long beep (stay for 1 second) is heard when temperature during 89.6°F~99.5°F (34°C~37.5°C) with green backlight displayed. 4 times Bi-bi is heard when temperature higher than 99.5°F/37.5°C with red backlight displayed. 2 times Bi-bi is heard first, then 4 times Bi-bi is heard when temperature higher than 109.2°F/42.9°C with icon  and red backlight displayed.

### 2. WATERBATH MODE:

Press button with a short beep to switch to waterbath mode .

The temperature will be changed without backlight until thermometer auto off after 5 minutes. Readings will be displayed when temperature during 41°F ~122°F (5°C ~50°C) within approximately 8 seconds.

### 3. AMBIENT MODE:

Press button with a short beep to switch to ambient mode .

The temperature will be changed without backlight until thermometer auto off after 5 minutes. Readings will be displayed when temperature during 14°F ~122°F (-10°C ~50°C) within approximately 8 seconds.

Read and record the temperature and time for reference. The reading will automatically be stored in memory, but it is recommended to read and record the temperature and time for later reference.

Press the ON/OFF button to turn the thermometer OFF. If you do not turn the thermometer OFF manually, the thermometer will shut off automatically in approximately 5 minutes.

## RECTAL USE (BODY MODE):

- Commonly used for babies, young children or when it is difficult to take an oral or underarm temperature.
- Lubricate probe tip with a water-soluble lubricant or petroleum jelly.
- Gently insert the probe tip no more than 1/2 inch (1.3 cm) into the rectum. Do not force the tip into the rectum if resistance is encountered.

- Clean the thermometer as recommended. See Care and Cleaning.
- The peak temperature should be reached in approximately 8 seconds.
- For rectal measurement, a fever is considered any temperature of 100.4° F (38°C) or higher.

## UNDERARM (AUXILIARY) USE (BODY MODE):

- Wipe underarm with a dry towel.
- Place probe tip in armpit and bring arm down against the body. The thermometer should be pointing upward in the same direction as your body if possible.
- The time required for an accurate underarm temperature may be 12 seconds or longer depending on placement of the probe and closure of the arm against the body.
- For armpit measurement, a fever is considered any temperature of 99° F (37.2°C) or higher.

## ORAL USE (BODY MODE): (FOR CHILDREN 4 YEARS AND OLDER)

- Do not drink hot or cold fluids, exercise or perform other activities that will raise or lower temperature readings when compared to your normal, average temperature. The mouth should remain closed up to 5 minutes before attempting a reading.
- Place the probe tip well under the tongue. Instruct children not to bite down on the probe.
- The peak temperature should be reached in approximately 10 seconds. Opening the mouth could result in a longer time for a reading.

- For oral measurement, a fever is considered any temperature of 100° F (37.8°C) or higher.

## TECHNICAL SPECIFICATIONS:

### Measurement Range:

Body Mode: 93.2°F ~109.2°F (34.0°C ~42.9°C) ± 0.2°F (0.1°C)

Waterbath Mode: 41°F ~122°F (5°C ~50°C) ± 2°F (1°C)

Ambient Mode: 14°F ~122°F (-10°C ~50°C)± 2°F (1°C)

### Measurement Accuracy:

Body Mode: Technical: +/- 0.2°F (0.1°C)

Waterbath Mode: Technical: +/- 2°F (1°C)

Ambient Mode: Technical: +/- 2°F (1°C)

### Operation Environment:

50°F ~ 104°F (10°C ~ 40°C), Humidity: 15-95% (non condensing) (complies with ASTM E1112)

### Storage Environment: -

13°F to 131°F (-25° C to 55° C), Humidity: 15-95% (non condensing)

## NASAL ASPIRATOR:

1. Insert cotton ball or wadded facial tissue in chamber to prevent mucus or bacterial transfer.
2. Place soft blue tip against child's nostril
3. Using long, slow breaths, use blue mouthpiece to suck out mucus.
4. Dispose of cotton ball or facial tissue after use.
5. Wash chamber with warm soap & water. Clean thin tube with a few drops of rubbing alcohol.
6. Firmly snap dry pieces together, inserting a new filter for next use.
7. Not dishwasher safe.

## CRADLE CAP BRUSH:

1. Wet baby's head with warm water. Use a small amount (1 tsp) of your favorite oil (baby, olive, coconut, etc) or tear-free baby shampoo and massage into baby's scalp.
2. Apply gentle pressure using a circular to massage the baby's head with the cradle cap brush; then rinse baby's head.
3. Rinse and repeat as necessary.
4. Wash the brush with soap and warm water and rinse thoroughly between uses. Store (bristles down) in an open area to air dry.

## SILICONE TOOTHBRUSH:

1. GAUZE SWAB – For children with no teeth, insert moistened gauze into blue opening and gently rub over gums to wipe clean once daily.
2. FINGER BRUSH – Once teeth are visible, remove blue silicone tip from toothbrush and place on index finger. Clean teeth with a gentle rubbing motion twice daily.
3. SILICONE TOOTHBRUSH – For children 1 year and older, use a small amount of children's toothpaste and brush teeth twice daily.

## MEDICINE SPOON:

1. Remove spoon tip from syringe.
2. Insert syringe into medicine and withdraw required amount.
3. Replace spoon tip onto syringe.
4. Easily dispense directly into baby's mouth or add small amounts to their favorite food.
5. Dishwasher Safe.

NOTE: Do not return remaining medicine to bottle.

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Consumer: Please retain all product information for future reference. Conforms to ASTM E1112. Actual product styling and colors may vary. Trademarks and copyrights are exclusively owned by BKD. Made in China.